1- DESCRIPCIÓN

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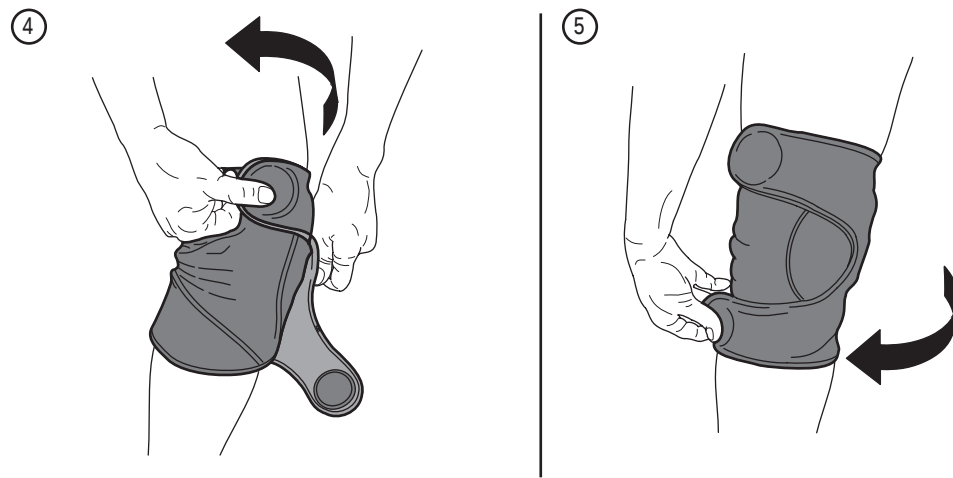
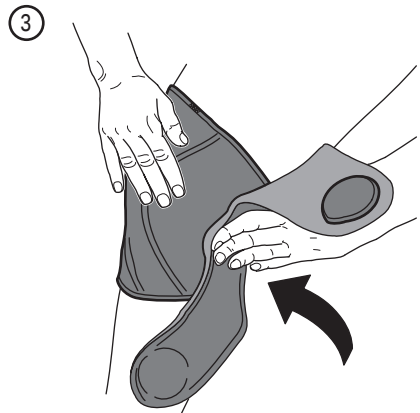
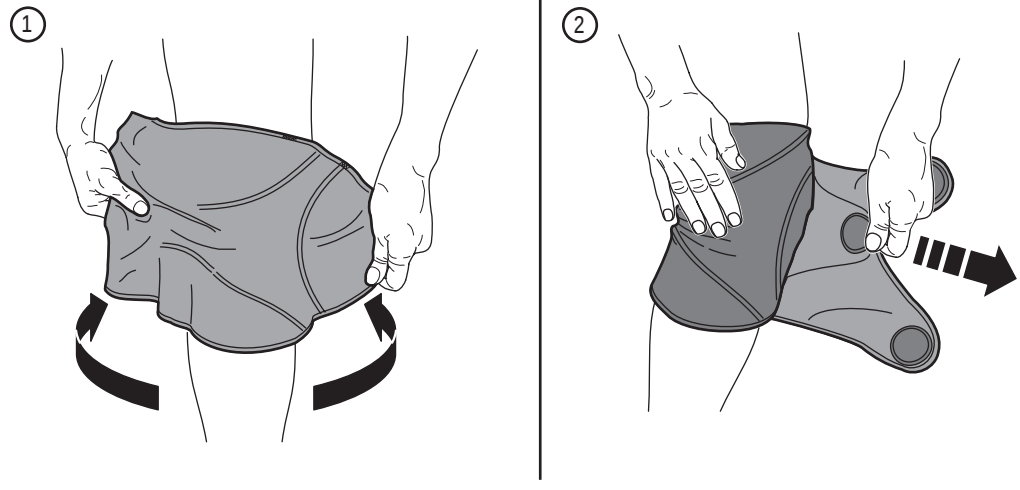
UK

#### 1- DESCRIPTION

This product is used as a protective support following muscular, articular or ligamentus injuries of the patellar region.

Two different materials were used in its construction;

- the inside is made of soft cotton velour that is breathable and does not cause skin irritation and
- the outside is a special nylon that permits the Velcro to grip and stay fastened.

Thanks to its construction, you can obtain an infinite possibility of adjustments superior to patella or inferior to the patella, depending on the degree of compression needed.

The Adjustable Knee Wrap was made in accordance with the primary requirements set under the European Directive 93/42/ECC and with the Italian law (LD 24th February 1997, No. 46) introducing the same EU Directive.

#### 2- INDICATIONS

Support and compression for light patellar trauma. Useful during rehab.

#### 2-1 INSTRUCTIONS

Apply directly on the skin following the directions on the drawings:

#### 3- SIDE EFFECTS AND CONTRA-INDICATIONS

Assuming that the products are used according to instructions and given that they are non invasive, they are no side effects affecting the organism as a whole. All braces and support, if overly tight, can cause discomfort due to compression. If you observe any change in your general state or deterioration of your condition, please consult your doctor. During a prolonged period of inactivity, it is recommended to reduce the tension on your device until you resume normal activities where you will reset the tension to the original level recommended to you.

Any modifications of these products will result in the annulment of the warranty along with the denial of any responsibilities on our behalf.

#### 4- MAINTENANCE AND CARE

- Before washing, replace Velcro's in their original position to prevent them from damaging other objects.
- Wash gently at lukewarm temperature.
- Do not bleach.

This product is not biodegradable. AT THE END OF ITS USABLE LIFE, DISCARD IT ALONG WITH YOUR NORMAL WASTE.

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