1- DESCRIPCIÓN

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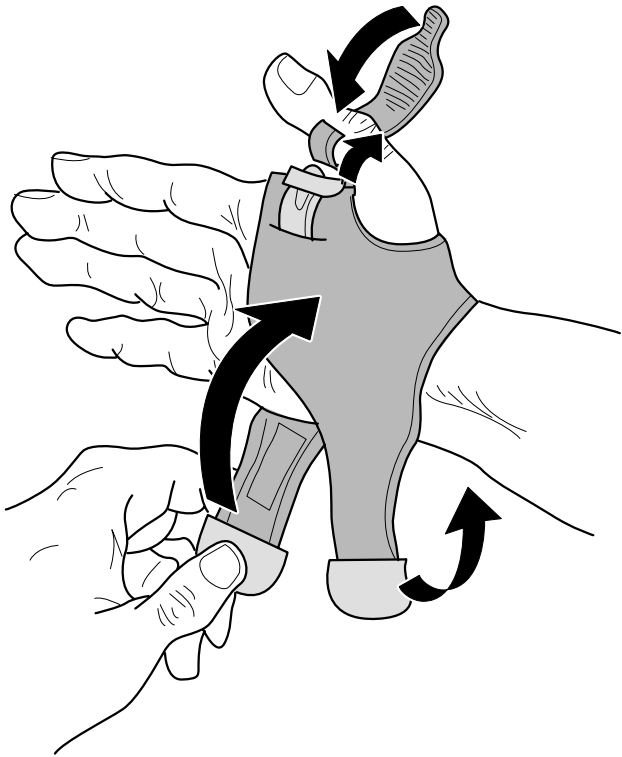
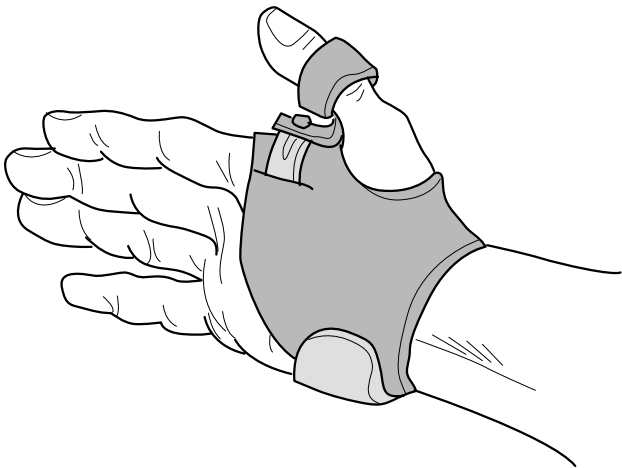
2

1- DESCRIPTION

The main features of the M-Brace Thumb splint are:

- an advanced fabric ensuring a very comfortable touch while allowing the skin to breathe;
- right and left models available;
- a structure which can be totally opened so that wearing the splint is easy and painless;
- RIZOFIX, the patented thumb block system immobilizes the thumb from the inner side, leaving the opposite side free. It is very useful and easy to adjust according to one's own needs.

Made in accordance with the primary requirements set under the European Directive 93/42/EEC on medical devices and the Italian corresponding law (LD 24th February 1997, No. 46) introducing the same Directive.

2- INDICATIONS

- Traumatic pathologies: distortions, carpal-metacarpal fractures, tendonitis.
- Degenerative pathologies: root arthrosis, carpal tunnel syndrome, arthrosis.
- Rehab phase: provides post-trauma and post-op tendon muscle relief.

2-1 INSTRUCTIONS

- Verify if right or left model needed.
- Open the M-Brace lace and put the support on the hand (fig. 1).
- Place the two Velcro stripes and the M-Brace lace around the thumb (fig. 2).

3- SIDE EFFECTS AND CONTRA-INDICATIONS

Assuming that the products are used according to instructions and given that they are non-invasive, they are no side effects affecting the organism as a whole. All braces and support, if overly tight, can cause discomfort due to compression. If you observe any change in your general state or deterioration of your condition, please consult your doctor. During a prolonged period of inactivity, it is recommended to reduce the tension on your device until you resume normal activities where you will reset the tension to the original level recommended to you.

Any modifications of these products will result in the annulment of the warranty along with the denial of any responsibilities on our behalf.

4- MAINTENANCE AND CARE

- No special maintenance is required.
- Fold the Wrist Support and firmly attach the Velcro pads to the surface.
- Wash gently at lukewarm temperature.
- Do not use any substances containing chlorine.
- Dry at low setting.
- Before washing remove the RIZOFIX System.

This device is non-biodegradable.

Even though it has been made with natural fabrics.

IT SHOULD NOT BE DISPERSED IN THE ENVIRONMENT BUT RATHER DISPOSED AS COMMON NON-BIODEGRADABLE MATERIALS.

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