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- AUSTRALIA -

**Distributed by
NEW HORIZONS Pty Ltd**
248 Haddie Street
Abbotsford
Victoria 3067
AUSTRALIA
Phone: +61 3 9419-3177 / Fax: +61 3 9419-5236
Email: info@bodyassist.com

ActivDay®

#503 - #504

Made in Italy by:

INDACO S.R.L.

Contrada Iscalunga snc
85020 Filiano - Potenza
Italy
www.activdaybrace.com

EN LUMBAR-SACRAL BRACE

Declaration of conformity (Directive 93/42/EEC, Annex VII)

INDACO srl - Hereby declares that the LUMBAR SACRAL BRACE complies with the provisions transposing the Directive on MEDICAL DEVICES (93/42/EEC).
GENERAL INFORMATION AND INSTRUCTIONSPage 3

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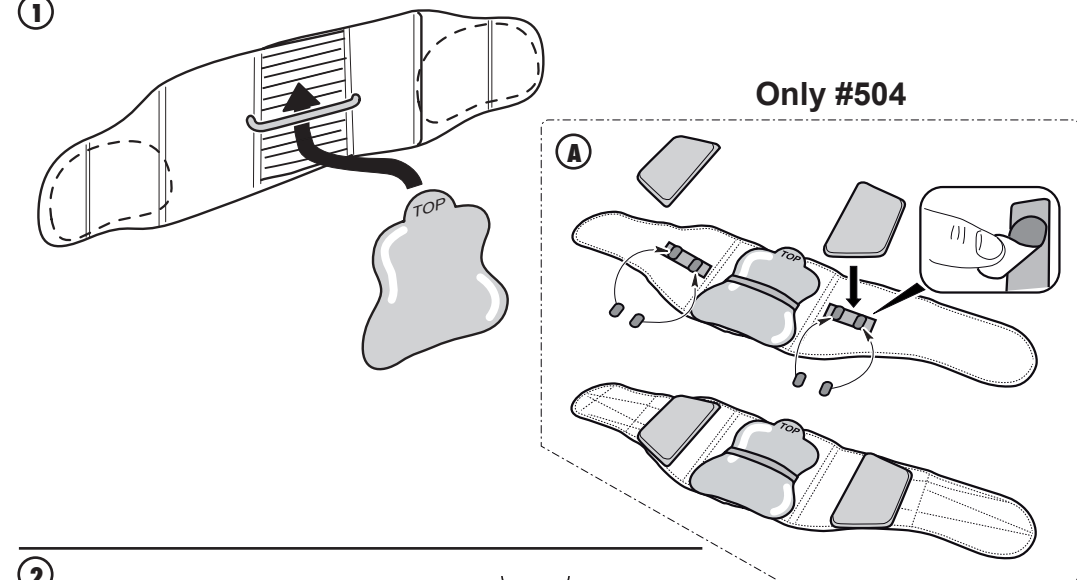
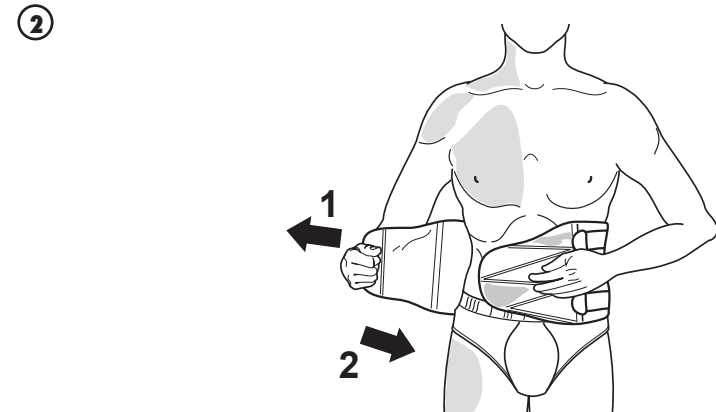
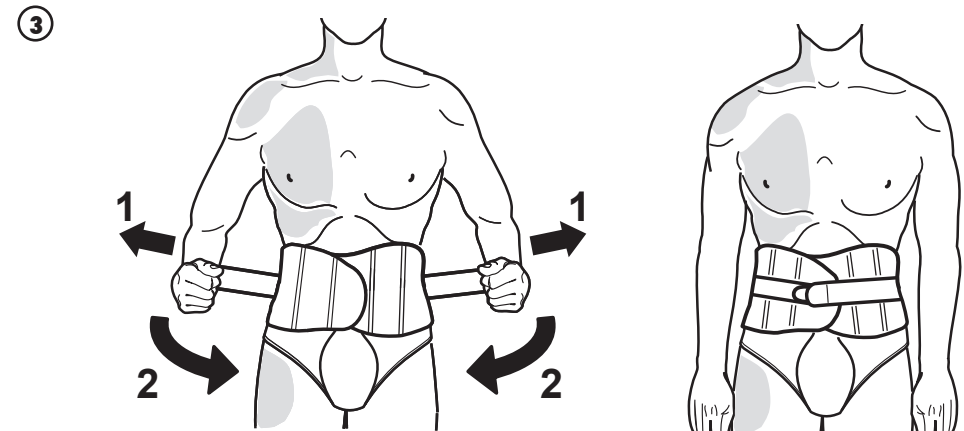
2

1- DESCRIPTION

This Lumbar-Sacral Brace acts as an immobilizing device for the lumbar-sacral region and its' primary function is to reduce pain caused by lumbago, sciatica and ischialgia.

Made with semi-rigid fabric, it enjoys a structure made with eight selfmodeling and flexible stays on which a number of rigid straps slide. When the locking system is pulled and secured at the front, the inside straps stiffen the entire structure, thus effectively immobilizing the lumbar-sacral region.

Two rigid thermo-formable panels for lumbar and abdominal application are included. These panels produce intra-abdominal pressure to reduce load on discs thus reducing pain and improving post-operative outcome when used with the corset in the appropriate position.

Includes two side panels mouldable to provide better support (#504).

The region supported extends from the sacroccygeal joint to T-9 vertebra.

Practical and easy to wear, this brace perfectly adapts to whatever size and body type, thus ensuring maximum support and comfort at the desired tension. Through a new locking technology, operated by means of two crossing bands, the closing system is totally user friendly and allows a wide range of tension settings.

This Lumbar-Sacral Brace is manufactured in accordance with the primary requirements set under the European Directive 93/42/EEC on medical devices and in accordance with the Italian law (LD 24th February 1997, No. 46) introducing the same EU Directive.

2- INDICATIONS

- Lumbago, sciatica and ischialgia.
- Lumbar-Sacral pain of other origin and nature.
- Disc complaints.
- Spondylo-arthritis.
- Osteoporosis related lumbar vertebrae complaints.
- Para-vertebral lumbar muscle spasm.
- Light trauma of the lumbar-sacral column.

2-1 HOW TO FIT CORSET

The LSO must be worn over a cotton undershirt. To apply it correctly, follow these instructions:

- 1 Insert the back panel into its elastic band as in figure 1.
#504: fasten the 4 "hook and loop fastener system" strips with the white side over the straps as shown in figure A.
- 2 Hold the left abdominal flap firmly with the left hand over the abdomen. Grasp the right flap with the right hand stretch the corset outwards, then wrap around the body using the "hook and loop fastener system" strips to attach flap securely onto first flap over the abdomen.
- 3 Verify the corset is fitted correctly by checking the position of the front flexible stays; they should be positioned equidistant from the navel.
- 4 Grasp the lower side straps and pull on them simultaneously until the required tension is achieved, then fasten them to the abdomen. Repeat this step to adjust the centre side straps (on applicable models) and finally the upper side straps.

3- SIDE EFFECTS AND CONTRA-INDICATIONS

Assuming that the products are used according to instructions and given that they are non invasive, they are no side effects affecting the organism as a whole. All braces and support, if overly tight, can cause discomfort due to compression. If you observe any change in your general state or deterioration of your condition, please consult your doctor. During a prolonged period of inactivity, it is recommended to reduce the tension on your device until you resume normal activities where you will reset the tension to the original level recommended to you.

Any modifications of these products will result in the annulment of the warranty along with the denial of any responsibilities on our behalf.

3

4- MAINTENANCE AND CARE

- No special maintenance is required.
- Remove the two panels.
- Fold the Lumbar-Sacral Brace and attach the "hook and loop fastener system" firmly to the surface.
- Wash gently at lukewarm temperature.
- Do not use any substances containing chlorine.
- Dry at low setting.
- Washing does not damage the product and it actually restores the elasticity.

This product is not biodegradable.

AT THE END OF ITS USABLE LIFE, DISCARD IT ALONG WITH YOUR NORMAL WASTE.

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