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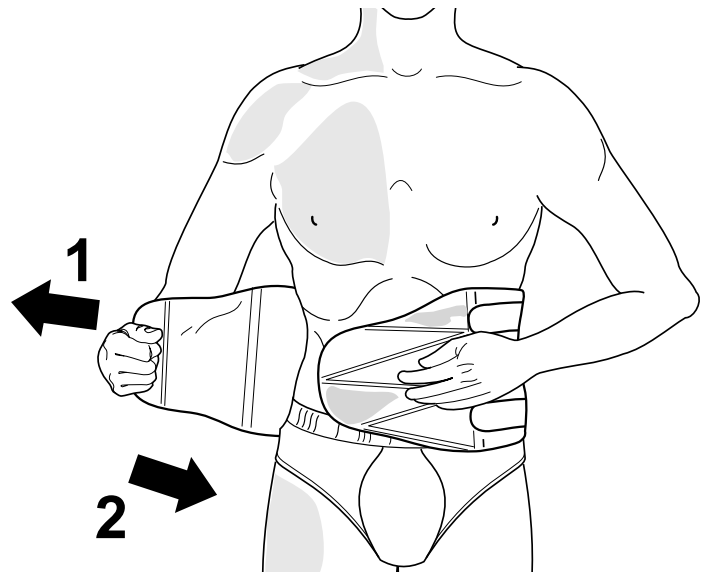
UK

1- DESCRIPTION

This LumbLock Lumbar-Sacral Brace acts as an immobilizing device for the lumbar-sacral region and its' primary function is to reduce pain caused by lumbago, sciatica and ischialgia.

Made with semi-rigid cotton fabric, it enjoys a structure made with eight selfmodeling and flexible steel stays on which a number of rigid straps slide. When the locking system is pulled and secured at the front, the inside straps stiffen the entire structure, thus effectively immobilizing the lumbar-sacral region.

Practical and easy to wear, this LumbLock brace perfectly adapts to whatever size and body type, thus ensuring maximum support and comfort at the desired tension. Through a new locking technology, operated by means of two crossing bands, the closing system is totally user friendly and allows a wide range of elastic tension settings. The Extra-LumbLock braces (573) carry two additional sleeves at the back where custom moldable, anodized aluminum stays have been inserted. Although the rigidity of these braces has been remarkably increased, the comfort characteristics remain unaltered.

This Lumbar-Sacral Brace is manufactured in accordance with the primary requirements set under the European Directive 93/42/EEC on medical devices and in accordance with the Italian law (LD 24th February 1997, No. 46) introducing the same EU Directive.

2- INDICATIONS

- Lumbago, sciatica and ischialgia.
- Lumbar-Sacral pain of other origin and nature.
- Disc complaints.
- Spondylo-arthritis.
- Osteoporosis related lumbar vertebrae complaints.
- Para-vertebral lumbar muscle spasm.
- Light trauma of the lumbar-sacral column.

2-1 HOW TO FIT IT

The LSO has to be worn over a cotton undershirt. To apply it correctly, follow these instructions:

- 1 Press one abdominal flap with the left hand over the abdomen, wrap the corset around the waist and hold the other flap with the right hand and stretch it away from the body. Finally, apply it over onto the stomach and Velcro it securely onto the other flap.
- 2 Verify the correct position of the corset by taking into consideration the position of the anterior flexible stays; they should be positioned equidistant from the navel.
- 3 Secure the corset by pulling contemporarily on the straps located on both sides of it and apply them onto the abdomen. You should start from the bottom straps and repeat this step securing the middle straps (on applicable models) and then the top ones.

3- SIDE EFFECTS AND CONTRA-INDICATIONS

Assuming that the products are used according to instructions and given that they are non invasive, they are no side effects affecting the organism as a whole. All braces and support, if overly tight, can cause discomfort due to compression. If you observe any change in your general state or deterioration of your condition, please consult your doctor. During a prolonged period of inactivity, it is recommended to reduce the tension on your device until you resume normal activities where you will reset the tension to the original level recommended to you.

Any modifications of these products will result in the annulment of the warranty along with the denial of any responsibilities on our behalf.

4- MAINTENANCE AND CARE

- No special maintenance is required.
- For POLLUX (the Extra LumbLock 573) Remove the two paraspinal aluminum stays.
- Fold the LumbLock Brace and attach the velcro pads firmly to the surface.
- Wash gently at lukewarm temperature.
- Do not use any substances containing chlorine.
- Dry at low setting.
- Washing does not damage the product and it actually restores the elasticity.

This product is not biodegradable.

AT THE END OF ITS USABLE LIFE, DISCARD IT ALONG WITH YOUR NORMAL WASTE.

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