1- DESCRIPCIÓN

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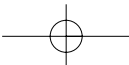
Distributed by
M.I.H. International inc.

153 Beaubien West - MONTREAL- Qc
H2V 1C5, CANADA
Tel: (514) 731-2617
Fax: (514) 731-5626
www.m-brace.com

Made in Italy by
INDACO s.r.l.

Contrada Iscalunga snc
85020 Filiano - Potenza
Italy

CLAVICLE SUPPORT

Declaration of conformity (Directive 93/42/EEC, Annex VII)

INDACO srl - Hereby declares that the CLAVICLE SUPPORT complies with the provisions transposing the Directive on MEDICAL DEVICES (93/42/EEC).
GENERAL INFORMATION AND INSTRUCTIONS.....Page 3

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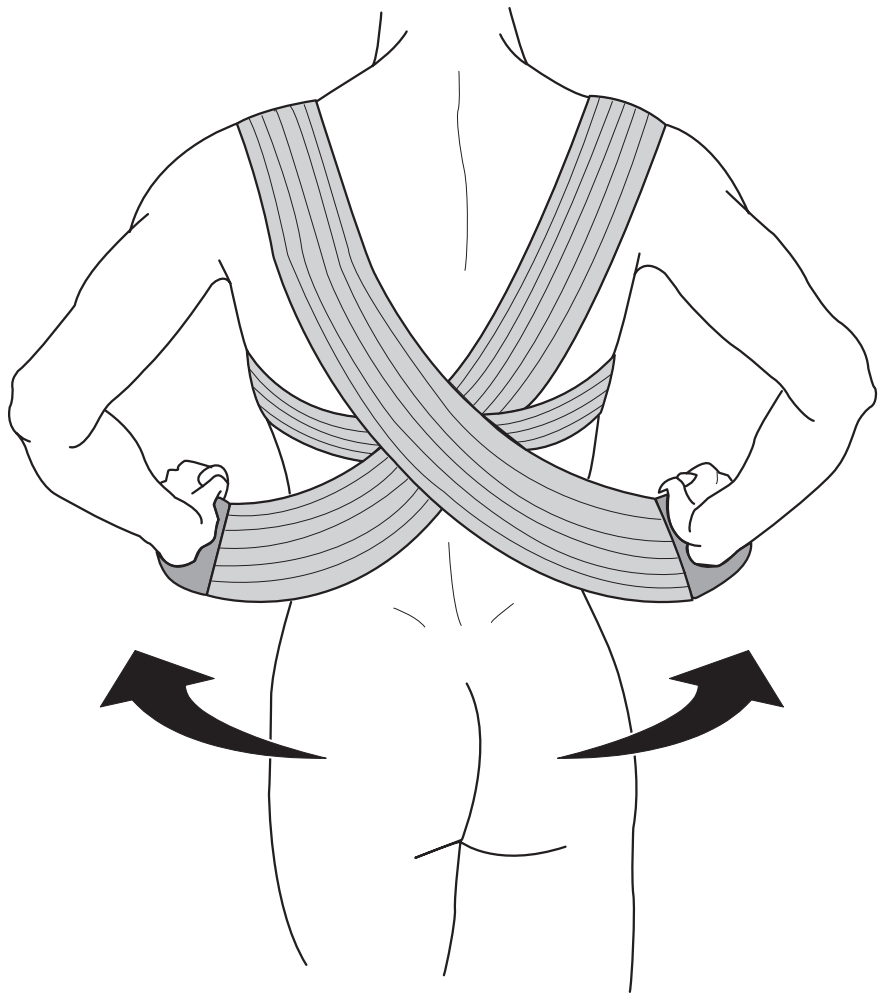
1- DESCRIPTION

Made of an elastic fabric, the CLAVICLE SUPPORT is comfortable and easy to put on. It is composed of two bands to be worn on the right and left shoulder, crossed behind the back and then locked together on the abdomen at the desired tightness. The constant support of the shoulders relaxes the pectoral muscles and improves posture. To ensure maximum comfort especially during extended use, this product was made with a breathable fabric. The CLAVICLE SUPPORT was made in accordance with the primary requirements set under the European Directive 93/42/EEC and with the Italian law (LD 24th February 1997, No. 46) introducing the same EU Directive.

2- INDICATIONS

Rigid or falling shoulders, bad posture and clavicle injuries.

2-1 INSTRUCTIONS

Wear over an undergarment.

The appropriate band has to be worn on the right and left shoulder making sure that the pads are on the front of the body (Right or left is indicated on each band). Cross behind the back and then lock together on the abdomen at the desired tightness.

3- SIDE EFFECTS AND CONTRA-INDICATIONS

Assuming that the products are used according to instructions and given that they are non invasive, they are no side effects affecting the organism as a whole. All braces and support, if overly tight, can cause discomfort due to compression. If you observe any change in your general state or deterioration of your condition, please consult your doctor. During a prolonged period of inactivity, it is recommended to reduce the tension on your device until you resume normal activities where you will reset the tension to the original level recommended to you.

Any modifications of these products will result in the annulment of the warranty along with the denial of any responsibilities on our behalf.

4- MAINTENANCE AND CARE

Before washing, replace Velcro's in their original position to prevent them from damaging other objects. Wash gently at lukewarm temperature. Do not bleach.

This product is not biodegradable.

AT THE END OF ITS USABLE LIFE, DISCARD IT ALONG WITH YOUR NORMAL WASTE.

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