136

USER MANUAL

V.02_2025/09

WRIST SPLINT
GOUTTIÈRE POUR LE POIGNET
TABLILLA DE MUÑECA
HANDGELENKSCHIENE
TALA DE IMOBILIZAÇÃO DE PULSO
IMMOBILIZZATORE DI POLSO

Digital version

ITALIANO

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ITALY

Made in Italy by
INDACO s.r.l.
Contrada Iscalunga snc
85020 Filiano - Potenza - Italy
www.m-brace.com

UNITED STATES

Distributed by
MIH International LLC
112 Capitol Trail Newark, DE
19711 - USA
NORTH AMERICA
Toll Free: 866-374-4050
Fax: 866-374-4051
Email: sales@m-brace.com
www.m-brace.com

CANADA

Distributed by
MIH International Inc.
285 Quinlan Lasalle,
QC H8R 3W4 CANADA
Phone: 514-731-2617
Fax: 514-731-5626
Toll Free: 1 866-374-4050
Email: contact@m-brace.com
www.m-brace.com

AUSTRALIA


Distributed by
Bodyassist Health and
Wellness Pty Ltd
269 Williamstown Road
Port Melbourne
VIC 3207 Australia
Phone: +61 3 9419-3177
Fax: +61 3 9419-5236
www.bodyassist.com

REF


136 size Regular Right
136 size Extra Right
136 size Regular Left
136 size Extra Left


1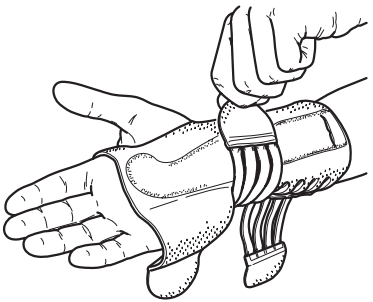



2

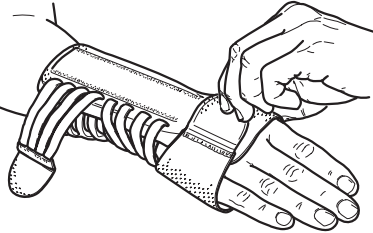3

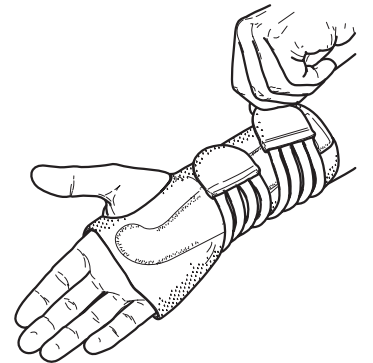4

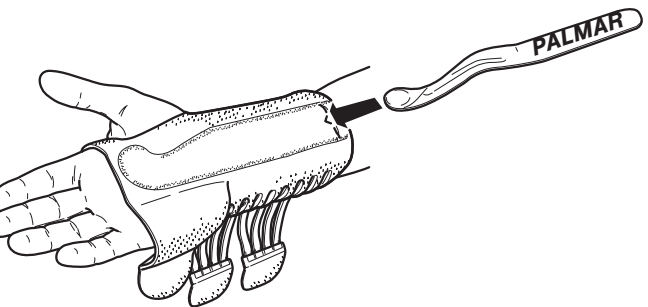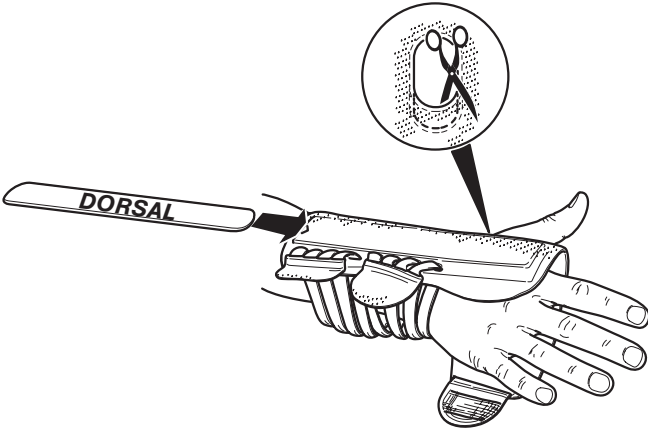ENGLISH

INTRODUCTION

The M-Brace wrist brace is designed to provide support, stability, and comfort to the wrist, helping to reduce pain and promote freedom of movement. Made from high-quality materials, it guarantees safe and reliable use in everyday life and during physical activity. Please read this manual carefully before use.

If you have any questions or require clarification, please contact your trusted retailer or write to contact@m-brace.com

PRODUCT FEATURES

- Specific fit: available in right or left versions for optimal comfort.
- Innovative fabrics: Interior: soft, breathable microfiber, gentle and comfortable against the skin. Exterior: high-strength nylon mesh with microscopic loops that allow the hook-and-loop fastening system to attach securely.
- Customized adjustment: the special construction allows for a wide range of adjustments based on the desired level of tension.
- Aluminum splints: two preformed and moldable splints to adapt to the patient's needs; they can be removed to adjust the degree of immobilization.

INDICATIONS

Traumatic conditions:

- Sprains Carpal
- Metacarpal fractures
- Tendinitis

Degenerative conditions:

- Radicular arthrosis
- Carpal tunnel syndrome
- Arthrosis

Rehabilitation phase:

- Post-trauma support
- Post-operative support

HOW TO WEAR

Apply the wrist brace directly to the skin, following the steps below:

- Insert your thumb through the oval opening (see figure).
- Close the brace in the following order: First the central strap, then the hand strap, and finally the forearm strap.
- If necessary, the oval opening for the thumb can be slightly enlarged on the advice of your doctor, for example to reduce compression in operated areas.
- Check that the hook-and-loop fastening system is correctly attached to the surface of the product and that it does not come into direct contact with the skin.

CARE AND MAINTENANCE

- Remove the splints before washing.
- Close the hook-and-loop fastening system to protect the brace and other garments.
- Wash gently by hand or in the washing machine on a warm cycle (max 30°C).
- Do not use substances containing chlorine or other bleaches.
- Air dry, away from direct heat sources or dryers.

DISCLAIMER

Do not self-diagnose or self-medicate. Always consult a doctor or qualified personnel before using the device. Follow their instructions and the information in this manual. If in doubt, consult your doctor, retailer, or our company.

RISKS AND CONTRAINDICATIONS

When used correctly, M-Brace products do not cause systemic side effects. However, excessive compression may cause local discomfort. If your condition worsens or your general condition changes, consult a doctor immediately*. During periods of prolonged inactivity, reduce the tension of the brace until you return to normal activities, at which point you can return the pressure to the recommended level.

WARRANTY The product warranty is governed by the

laws in force in the country of purchase. If necessary, first contact the retailer where you purchased the product. Before requesting warranty service, the product must be washed according to the instructions. The warranty may be limited or excluded if the product care and use instructions are not followed.

- The warranty does not cover cases of:
 - Use not in accordance with the instructions in the manual.
- Failure to follow the advice of specialized personnel*.
- Unauthorized modifications to the product.

NOTIFICATION REQUIREMENT

In accordance with EU Regulation 2017/745 on medical devices, the user must report any serious incidents that occur during use of the product to both the manufacturer and the competent authority of the Member State in which they reside.

DISPOSAL

At the end of its useful life, the product must be disposed of in accordance with local regulations.

**Specialist personnel refers to any person who, according to national or institutional regulations and other applicable guidelines, is authorized to fit orthopedic aids and provide instruction on their use. As manufacturers, we strongly recommend that such persons have orthopedic technology training or comparable skills and have access to orthopedic technology infrastructure.*

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