139

USER MANUAL

V.02_2025/09

EPICONDYLITIS BRACE
COUDIÈRE POUR ÉPICONDYLITE
REFUERZO PARA EPICONDILITIS
EPICONDYLITIS-BANDAGE
SUPORTE PARA EPICONDILITE
FASCIA PER EPICONDILITE

Digital version

ITALIANO

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ITALY

Made in Italy by
INDACO s.r.l.
Contrada Iscalunga snc
85020 Filiano - Potenza - Italy
www.m-brace.com

UNITED STATES

Distributed by
MIH International LLC
112 Capitol Trail Newark, DE
19711 - USA
NORTH AMERICA
Toll Free: 866-374-4050
Fax: 866-374-4051
Email: sales@m-brace.com
www.m-brace.com

CANADA

Distributed by
MIH International Inc.
285 Quinlan Lasalle,
QC H8R 3W4 CANADA
Phone: 514-731-2617
Fax: 514-731-5626
Toll Free: 1 866-374-4050
Email: contact@m-brace.com
www.m-brace.com

AUSTRALIA


Distributed by
Bodyassist Health and
Wellness Pty Ltd
269 Williamstown Road
Port Melbourne
VIC 3207 Australia
Phone: +61 3 9419-3177
Fax: +61 3 9419-5236
www.bodyassist.com

REF


139 size Regular
139 size Extra

1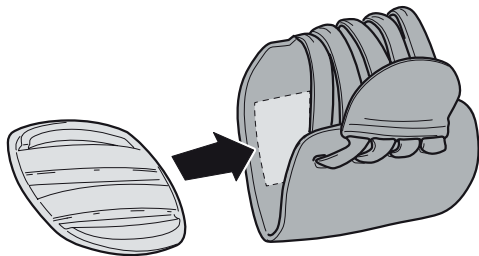


2

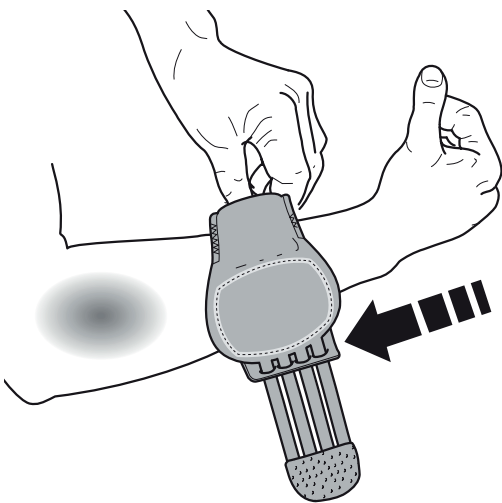3

4

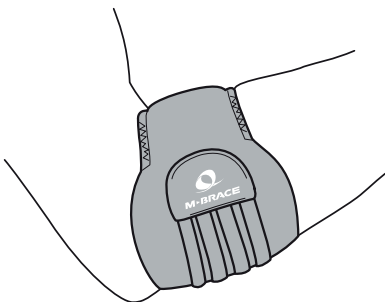ENGLISH

INTRODUCTION

The M-Brace epicondylitis brace is designed to provide targeted support and protection in cases of tendon inflammation at the epicon-dyle (tennis elbow/golfer's elbow), muscle pain and joint discomfort caused by minor trauma, functional overload or excessive strain of the forearm. Made from high-quality materials, it ensures safe and reliable use in everyday life and during work or sports activities.

Before use, carefully read this manual. For any questions or clarifications, please contact your trusted retailer or write to contact@m-brace.com

PRODUCT FEATURES

- Innovative fabrics: Inner layer: soft, breathable microfiber, gentle and comfortable against the skin. Outer layer: high-resistance nylon mesh with microscopic loops that allow the hook-and-loop fastening system to attach securely.
- Personalized adjustment: The special construction allows for a wide range of adjustments according to the desired level of tension.
- Ergonomic pad covered with 100% cotton on the skin-contact side.

INDICATIONS

- Tendon pathologies of the elbow:
- Lateral epicondylitis ("tennis elbow")
 - Medial epicondylitis/epitrochleitis ("golfer's elbow")

Overload and repetitive microtrauma:

- Repetitive manual work activities.
- Sports activities with intensive use of the forearm (racket sports, tools, weights, etc.)

Rehabilitation phase:

- Support during recovery phases after acute painful episodes.
- Support during post-treatment or post-operative phases, according to medical advice.

HOW TO WEAR

Apply the brace directly to the skin, following the steps below:

- Attach the pad inside the brace using the dedicated hook-and-loop fastener. (Figure 1).
- Insert the arm into the brace (Figure 2).
- Tighten the fastening system to the desired level of tension (Figure 3).

CARE AND MAINTENANCE

- Remove the splints before washing.
- Close the hook-and-loop fastening system to protect the brace and other garments.
- Wash gently by hand or in the washing machine on a warm cycle (max 30°C).
- Do not use substances containing chlorine or other bleaches.
- Air dry, away from direct heat sources or dryers.

DISCLAIMER

Do not self-diagnose or self-medicate. Always consult a doctor or qualified personnel before using the device. Follow their instructions and the information in this manual. If in doubt, consult your doctor, retailer, or our company.

RISKS AND CONTRAINDICATIONS

When used correctly, M-Brace products do not cause systemic side effects. However, excessive compression may cause local discomfort. If your condition worsens or your general condition changes, consult a doctor immediately*. During periods of prolonged inactivity, reduce the tension of the brace until you return to normal activities, at which point you can return the pressure to the recommended level.

WARRANTY

The product warranty is governed by the laws in force in the country of purchase. If necessary, first contact the retailer where you purchased the product. Before requesting

warranty service, the product must be washed according to the instructions. The warranty may be limited or excluded if the product care and use instructions are not followed.

- The warranty does not cover cases of:
- Use not in accordance with the instructions in the manual.
 - Failure to follow the advice of specialized personnel*.
 - Unauthorized modifications to the product.

NOTIFICATION REQUIREMENT

In accordance with EU Regulation 2017/745 on medical devices, the user must report any serious incidents that occur during use of the product to both the manufacturer and the competent authority of the Member State in which they reside.

DISPOSAL

At the end of its useful life, the product must be disposed of in accordance with local regulations.

"Specialist personnel refers to any person who, according to national or institutional regulations and other applicable guidelines, is authorized to fit orthopedic aids and provide instruction on their use. As manufacturers, we strongly recommend that such persons have orthopedic technology training or comparable skills and have access to orthopedic technology infrastructure."

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