40 - 41

USER MANUAL

V.03_2025/10

PATELLA STABILIZER
ORTHÈSE DE STABILISATION DE LA ROTULE
ESTABILIZADOR DE RÓTULA
PATELLA-STABILISATOR
ESTABILIZADOR DE RÓTULA
STABILIZZATORE DI ROTULA

Digital version

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AUSTRALIA

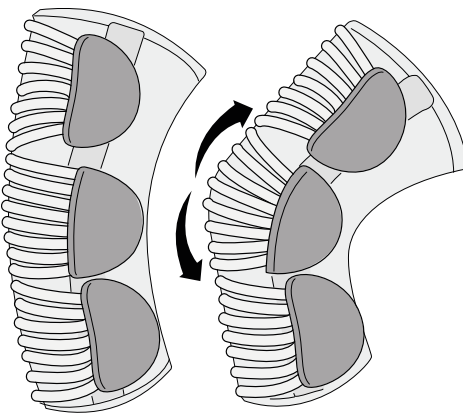
Distributed by
Bodyassist Health and
Wellness Pty Ltd
269 Williamstown Road
Port Melbourne
VIC 3207 Australia
Phone: +61 3 9419-3177
Fax: +61 3 9419-5236
www.bodyassist.com

REF


40 size XS	41 size XS
40 size S	41 size S
40 size M	41 size M
40 size L	41 size L
40 size XL	41 size XL
40 size XXL	41 size XXL
40 size XXXL	41 size XXXL
40 size 4XL	41 size 4XL

1

2

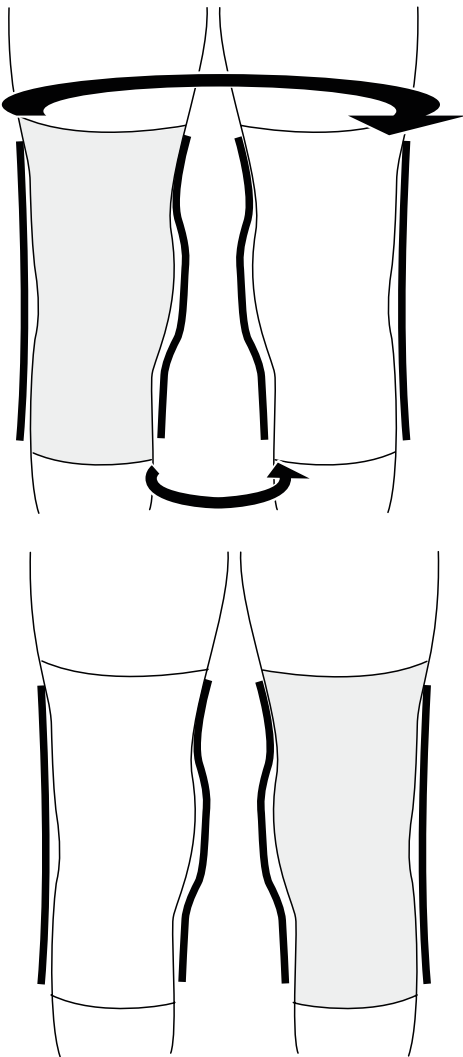ENGLISH

INTRODUCTION

The M-Brace knee brace is designed to provide support, stability and comfort to the knee, helping to reduce pain and promote freedom of movement. Made from high-quality materials, it guarantees safe and reliable use in everyday life and during physical activity.

Please read this manual carefully before use. If you have any questions or require clarification, please contact your trusted retailer or write to contact@m-brace.com

PRODUCT FEATURES

- Ambidextrous: can be used on either the right or left knee.
 - Targeted support: ideal for muscle, joint or tendon injuries in the patellar region.
 - Quality materials: made exclusively from breathable, hypoallergenic fabrics, free from neoprene; the area in contact with the skin is made from 100% soft cotton.
 - Exclusive locking system: allows pressure to be adjusted in three different areas for personalised support.
 - Patellar stabilisation: two C-shaped pressure pads join around the patella, ensuring stability and protection.
- Support rods:
- Products 40 and 41: two super-light, low-profile TPU rods to keep the knee brace in place.
 - Product 41: two multi-centre aluminium rods (lateral and medial) for greater support in case of collateral ligament instability.

INDICATIONS

Traumatic conditions

- Patellar instability or subluxation
- Patellofemoral osteoarthritis
- Patellar fractures
- Post-operative treatment (in the case of associated conditions: patella + mild collateral ligament instability – product 41 only

Rehabilitation phase

- Passive support during functional recovery
- Knee protection during sporting activities

HOW TO WEAR

Apply the knee brace directly to the skin following the steps below:

- Insert the side splints;
- For product 41, position the medial and lateral splints correctly (Figure 2);
- Open the hook-and-loop fastening system completely.
- Put on the knee brace with the label facing upwards. Position it so that the C-shaped pressure pads surround the kneecap.
- Close the three fastening straps by pulling both ends of the hook-and-loop fastening system simultaneously in the following order: first the central strap, then the lower strap and finally the upper strap.

CARE AND MAINTENANCE

- Remove the rods before washing.
- Close the hook-and-loop fastening system to protect the brace and other garments.
- Wash gently by hand or in the washing machine on a warm cycle (max 30°C).
- Do not use substances containing chlorine or other bleaching agents.
- Air dry, away from direct heat sources or tumble dryers.
- Washing does not damage the product, but maintains its elasticity and effectiveness.

DISCLAIMER OF LIABILITY

Do not self-diagnose or self-medicate. Always consult a doctor or qualified personnel before using the device. Follow their instructions and the information provided in this manual. If in doubt, consult your doctor, retailer or our company.

RISKS AND CONTRAINDICATIONS

When used correctly, M-Brace products do not cause systemic side effects. However,

excessive compression may cause local discomfort. If your condition worsens or your general health changes, consult a doctor immediately. During periods of prolonged inactivity, reduce the tension of the brace until you return to normal activities, at which point you can return the pressure to the recommended level.

WARRANTY

The product warranty is governed by the laws in force in the country of purchase. If necessary, please contact the retailer where you purchased the product first. Before requesting warranty service, the product must be washed according to the instructions. The warranty may be limited or excluded if the product care and use instructions are not followed.

The warranty does not cover the following cases:

- Use not in accordance with the instructions in the manual.
- Failure to follow the advice of specialised personnel*.
- Unauthorised modifications to the product.

NOTIFICATION OBLIGATION

In accordance with EU Regulation 2017/745 on medical devices, the user must report any serious incident that occurs during use of the product to both the manufacturer and the competent authority of the Member State in which they reside.

DISPOSAL

At the end of its useful life, the product must be disposed of in accordance with local regulations.

**Specialised personnel refers to any person who, according to national or institutional regulations and other applicable guidelines, is authorised to fit orthopaedic aids and provide instruction on their use. As manufacturers, we strongly recommend that such persons have orthopaedic technology training or comparable skills and have access to orthopaedic technology infrastructure.*

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