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#46

Distributed by
MIH International LLC
112 Capitol Trail
Newark, DE 19711 - USA

North America Toll Free
Tel: 866-374-4050 - Fax: 866-374-4051
Email: sales@m-brace.com
www.m-brace.com

Made in Italy by
INDACO s.r.l.
Contrada Iscalunga snc
85020 Filiano - Potenza
Italy

EN OA KNEE BRACE WITH RANGE OF MOTION

Declaration of conformity (Directive 93/42/EEC, Annex VII)

INDACO srl - Hereby declares that the #46 brace complies with the provisions transposing the Direc-tive on MEDICAL DEVICES (93/42/EEC).

GENERAL INFORMATION AND INSTRUCTIONSPage 3

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
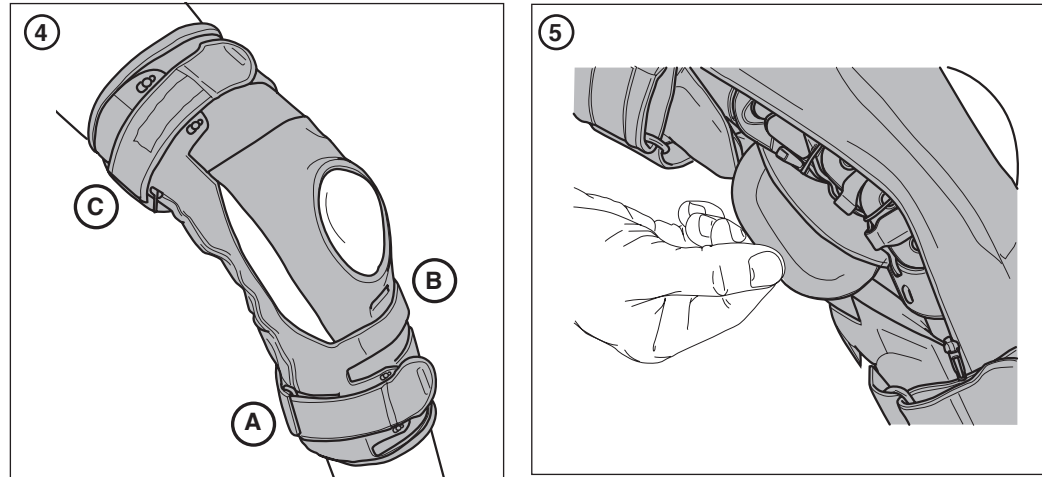
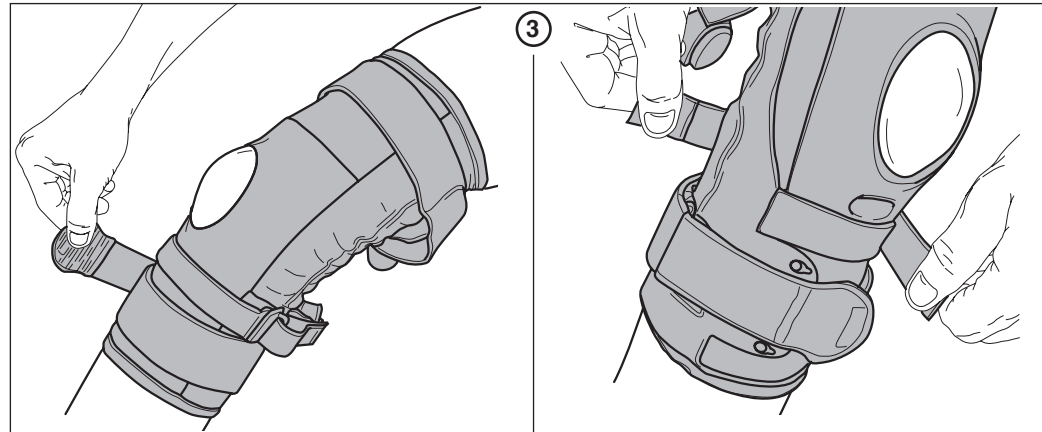
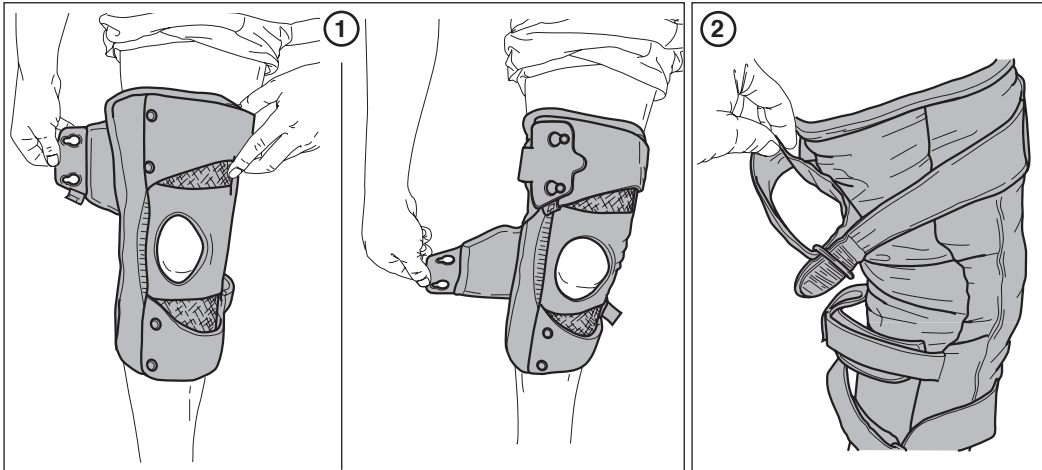
2

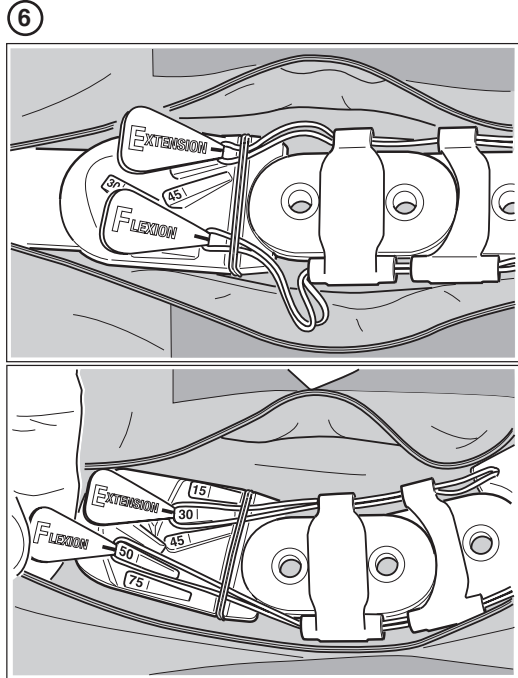(EN)

Visit a knee specialist, such as an orthotist or orthopaedic, who will assess your knee pain and decide what type of knee brace will work best. It is recommended that you work with a physical therapist or other healthcare professional and practice fitting your brace until you are comfortable with the procedure.

1- DESCRIPTION

In your quest to live an active lifestyle, the #46 knee brace brings your comfort to a new level with its elegant and light design. Available for the right or left knee, this brace alleviates knee pain by reducing the weight on your medial or lateral compartment by using the three point pressure system. It is composed of two aluminum reinforced semi-rigid straps that surround the distal region (above the knee) and the proximal region (below the knee). These straps are to be custom moulded to allow for an optimal torsion support. Additionally, these straps are bound together by a patented articulated aluminum support that is easily adjustable for flexion and extension and accessible within a zipped pocket. The patient can adjust the amount of lateral pressure applied by choosing between two different buttress (cushion) thicknesses. The high surface of the brace's sleeve allows for comfortable support around the affected joint. The patellar support on this brace is easy and simple to use and allows for a straight forward fitting of the product. Should you need to mould the brace to certain specifications or simply to wash the sleeve, the articulated support could be easily removed by unzipping the brace (see instructions). This brace is produced according to regulations 93/42/CEE referring to medical devices and Italian regulations (Feb 1997, no 46).

2- INDICATIONS

Use this knee brace for the following indications:

- Mild to moderate medial compartment OA without instability
- Meniscus procedures
- Chondral defect procedures
- Moderate lateral compartment OA without instability; it is possible to reduce the load on the lateral compartment by using a "right" brace on a left leg or a "left" brace on a right leg.

3

3 INSTRUCTIONS

IMPORTANT INFORMATION:

- If your pathology involves the MEDIAL compartment, use a brace marked "right" for the right leg or marked "left" for the left leg
 - If your pathology involves the LATERAL compartment, use a brace marked "right" for the left leg or marked "left" for the right leg
1. Fit directly on the skin with the white label going towards the top of the leg, loosen all the Velcros, following the instructions.
 2. On the first fitting, it is possible to adjust the straps on the back side of the brace to adapt to size.
 3. Start by tightening the semi-rigid straps. For future fittings you will only need to tighten the Velcros on the front side of the brace.
 4. Always tighten the brace from bottom up as per the drawing A then B then C
 5. Insert one or two pads as illustrated depending on your doctor prescription.
 6. To adjust flexion and extension as per your doctor's indications, open the access to lateral hinges and you will find two straps. A red one marked "flexion" and a grey one marked "extension". Pull the red flexion strap and attach it to the hook marked with the desired degree. Proceed the same way with the grey extension strap.
 7. It is possible to remove the hinges by opening the zipped compartment, removing the flexion and extension straps from their respective hooks, bending (flexing) the hinge completely and pulling out the hinge as per drawings below. To reassemble, proceed the opposite way.

4 - CONTRA-INDICATIONS AND SIDE EFFECTS

Following the assumption that the instructions have been followed, and given that this is not an invasive product, there have not been any observed side effects. As with any brace, if excessively tight it can cause local discomfort. If this happens, loosen the brace and consult your health professional. Any modification to this product voids the warranty and exempts us of any responsibility resulting from product damage or bodily injuries.

5 - CARE AND MAINTENANCE

- No special maintenance is required
- Washing this product restores its elasticity. Wash at lukewarm temperature
- Dry at low temperature
- Do not use bleach
- Reattach all Velcros in their position before washing

This product is not biodegradable and, therefore, it should be disposed of in the regular trash at the end of its useful life.

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